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How to use U.P. Fiber Splint

1. Orthodontic retainer

① To select U.P. Fiber Splint #1 (width 1 mm). ② To prepare the teeth for bonding using your standard bonding procedure. ③ To apply a thin layer of a paste composite or medium or high viscosity flowable resin at the level of the interproximal contacts. ④ To wet U.P. Fiber Splint with unfilled bonding agent and adapt the splint into the flowable resin, U.P. Fiber Splint must be closely laminated against the tooth like a bonded bracket when applying the splint. ⑤ To hold the adapted splint against the tooth with your finger and use an instrument to place the splint into the interproximal contacts closely against the adjacent teeth. To continue one tooth at a time until the entire U.P. Fiber Splint is placed. ⑥ To remove extra composite or flowable resin and cure after checking whether the splint is closely adapted against the teeth and deep into the interproximal contacts. ⑦ Using applicator brushes or instruments etc. to apply a thin coat of a flowable resin over the splint and cure.


2. Periodontal splint Option

1) Channel preparation

To prepare a channel at the level of the interproximal contacts to make the splint more resistant to debonding and maximize the longevity and comfort of the splint, more esthetic.

2) If not making a channel preparation, cut small depressions towards the distal of the fossa of the terminal teeth. It is to tuck the terminal ends of the splint into these depressions in order to prevent exposure of the fibers over time.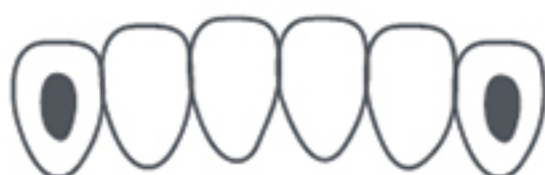

Measure the teeth and cut the U.P. Fiber Splint

- 1) To make a pattern by closely adapting a strip of tinfoil or dental floss to the teeth and into the contacts for measure.
- 2) To remove the U.P. Fiber Splint from the package and cut to the measured length on a clean place.

Prepare bonding

- 1) Prior to cleaning of the teeth, to clean the teeth using a diamond bur etc. to roughen the enamel. In order to clean the interproximals using finishing strips.
- 2) To prepare the teeth for bonding in standard manner.

Option

Using block-out resin : to apply a block-out resin to the area to be splinted after acid etching. This stabilizes the teeth during splinting and makes clean up easier.


Apply composite in labial interproximals

- 1) To apply a small of tooth shade filled composite to the labial interproximals in order to prevent teeth rotating and debonding.
- 2) Don't force the composite to the lingual surface. Cure.

Wet the U.P. Fiber Splint with a bonding agent

- 1) To wet the U.P. Fiber Splint with an unfilled bonding adhesive and blot off the excess with a gauze. It is better to touch the wetted splint with powder free gloves or clean fingers.
- 2) Don't cure yet.


Caution: Don't use a self-etching bonding agent, one bottle bonding system to contain dentin primer to wet the U.P. Fiber Splint.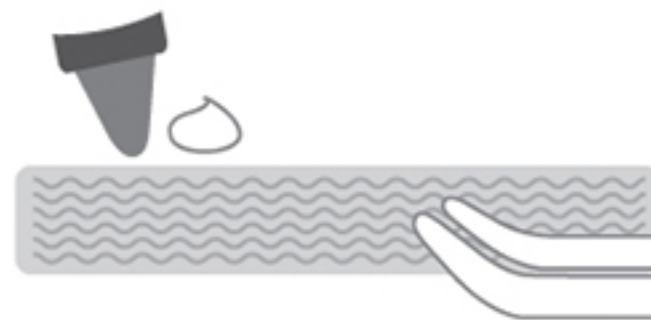

Apply filled composite to the teeth

- 1) To apply a medium viscosity composite resin at the level of the interproximal contact area.
- 2) If using a filled composite, flatten the composite to approximately 0.5 mm thick and contour to the surface of the teeth including within the interproximal contacts.

3) Do not cure yet.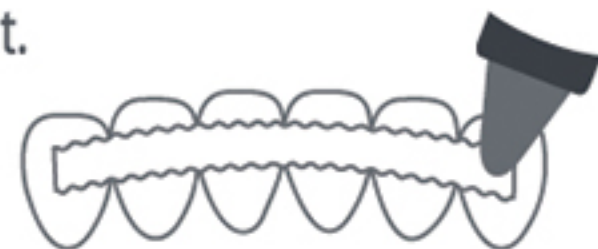

Adapt the U.P. Fiber Splint to tooth surface and interproximals

- 1) Starting at one end, closely adapt the U.P. Fiber Splint against the tooth surface like below figure 1 and deep into the interproximal contacts using a thin instrument like figure 3 and 4.
- 2) To continue tooth by tooth until entire length is adapted like figure 2. Don't cure.
- 3) If a channel preparation was made, press the U.P. Fiber Splint through the composite and adapt it to the floor of the preparation.

Figure 1

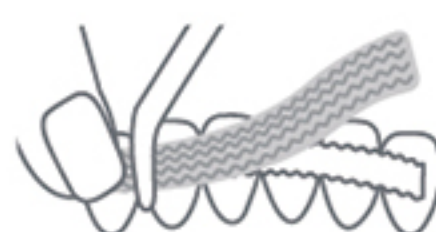
Figure 2

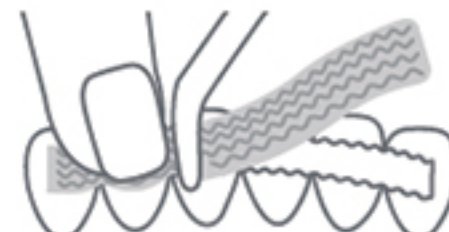
Figure 3

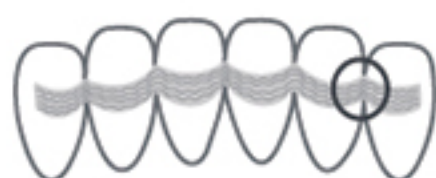
Figure 4

Remove excess composite

- 1) To remove excess composite with an instrument prior to curing.

Light-cure the splint

- 1) To cure one tooth at a time for 45 to 60 seconds if using a LED light cure unit, curing time is enough only for 30 ~ 40 seconds.

Cover the U.P. Fiber Splint with a flowable resin and cure

- 1) To cover the splint with a flowable resin.
- 2) Prior to curing, to make the covering layer as smooth as possible.
- 3) To apply enough resin over the terminal ends of the splint to minimize the chance of fiber exposure due to wear.
- 4) To avoid slump of flowable resin, apply multi thin layer of flowable resin over the splint.
- 5) To cure for 5 seconds to stop slumping after applying each layer.
- 6) To perform a full cure once all layers are applied.
- 7) If a channel preparation is used, cover the splint with a filled composite resin.

Check occlusion and finishing, polishing

- 1) To check occlusion.
- 2) To remove excess composite resin and polish, finish using a polishing paste.

Note

- 1) Never use a self-etching bonding system to wet the U.P. Fiber Splint, also never use a one bottle bonding system that contains dentin primer. If using a 5th generation bonding system, have to blow oil-free air to splint for a minute to remove acetone or acid in bonding agent.
- 2) In order to prevent problem of exposed fibers, use a flowable resin as the smoothing layer and lightly polish with a rubber cup.