

• Application method

1. Isolate the teeth in the desired area.
2. Spread separating materials and dry that.
3. Open the INO MULTI-GEL and install the enclosed on the syringe.
4. Inject the INO MULTI-GEL on the model.
5. Light curing the injected gel by using the UV lamp.
6. After the INO MULTI-GEL hardens, separate that from the model.

• Caution

1. Prohibit using of other purpose.
2. Users use the product after familiarizing themselves with how to use it.
3. Wear acid resistant protective gloves, protective clothing and goggles.
4. Be careful not contact with skin, eyes, mucous, and clothing.
5. In the event of accidental contact with the eyes or skin, wash it with a lot of water immediately and take medical measures.
6. Don't use expired products.

• Storage

1. Keep the syringe closed after use.
2. Avoid high temperature and direct sunlight.

• Contents

MG 14 RED 14ml x 1 syringe, Disposable tips

MG 14 RED 5 14ml x 5 syringes, Disposable tips

MG 14 PINK 14ml x 1 syringe, Disposable tips

MG 14 PINK 5 14ml x 5 syringe, Disposable tips

• Shelf-life

3 years from the date of manufacture

• Symbols



LOT




expiration period



Reference



manufacturer

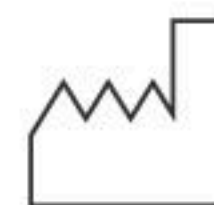

Date of Manufacture



Refer to user manual



Caution